

LUSON Simulation Center Equipment Do's and Don'ts

Do's

1. Handle all equipment with care
2. Leave all full body simulators as you would leave your patient
3. Make sure all equipment is turned off before you leave the room
4. Report any broken equipment to the Simulation Center Coordinator
5. If a manikin requires cleaning, notify the Simulation Center Coordinator
6. Lab staff should unplug all equipment prior to cleaning
7. Lab staff should clean all equipment according to the cleaning protocol

Don't

1. Do Not use any of the following on any equipment within the Simulation Center unless approved by manufacturer
 - a. Acetone
 - b. Iodine
 - c. Magic erasers
 - d. Any staining substance
 - e. Goo-Gone
 - f. Citrus based cleaners
 - g. Pen and marker
 - h. Food dye
2. Do Not have any food or uncapped drink in the Simulation Center
3. Do Not ever place liquid inside the manikin cavity or on the inside of the B/P arms
4. Do Not remove or transfer any of the full body simulators without permission from the Simulation Center Coordinator
5. Do Not remove any equipment from the Simulation Center